Goldpad[®]

Anjio Ped

**Pressure Assisted Dressing
(NON - ALLERGIC)**

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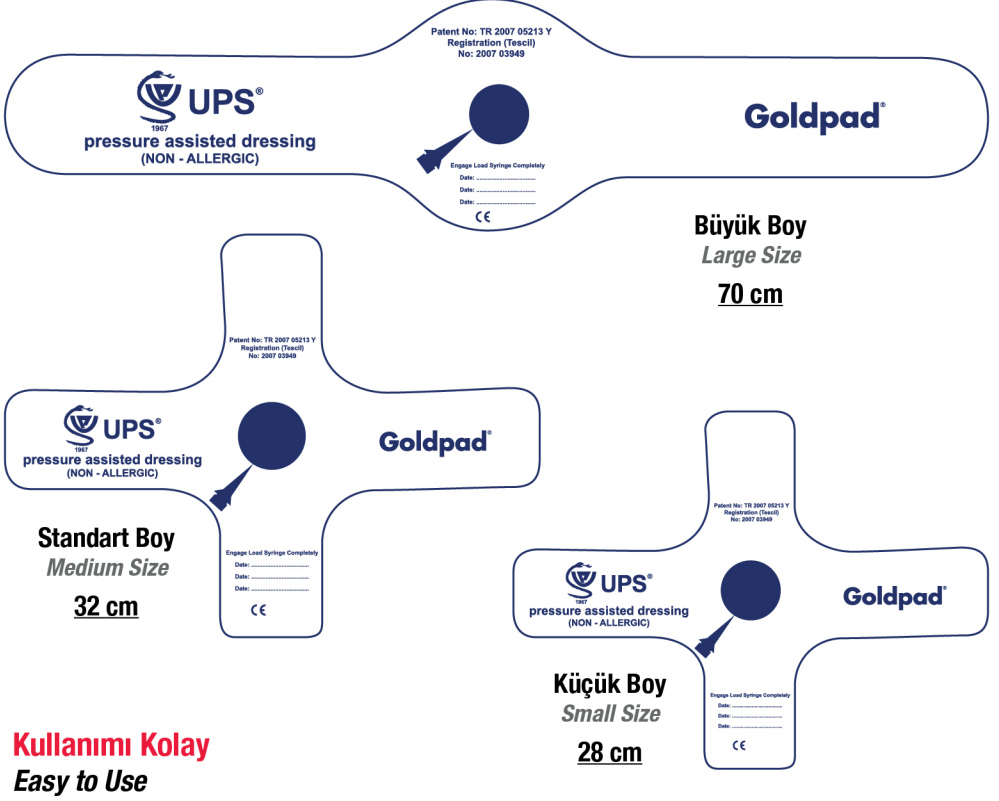
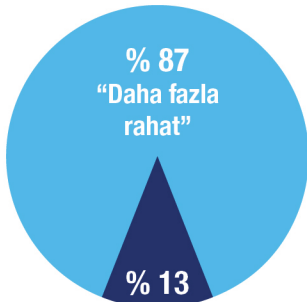
REAL PRESSURE

Goldpad delivers adjustable active compression and enables immediate pressure adjustment. Maintains consistent pressure on the site during patient recovery as well as patient positioning and transport. Provides site management control for non-compliant patient. Facilitates site assessment through the clear window without removing the device. Can be used as sterile dressing to protect the site from contamination.


Goldpad 28 cm Bulb Pressure

Air Pressure	Average Power	Maximum Power	Minimum Power
0	0.12 lbs	0.24 lbs	0.09 lbs
20	4.89 lbs	6.93 lbs	0.74 lbs
30	11.59 lbs	14.85 lbs	8.83 lbs
40	18.74 lbs	22.55 lbs	15.20 lbs

REAL COMFORT

Goldpad is comfortable for the patient. Maintains consistent pressure on the site when obtaining hemostasis. In the clinical study, 87% of the patient who had undergone a catheterisation procedure in the past indicated that Goldpad with MAT was "much more comfortable" than any previous procedure. Goldpad adheres to the patient while inflated regardless of patient anatomy.

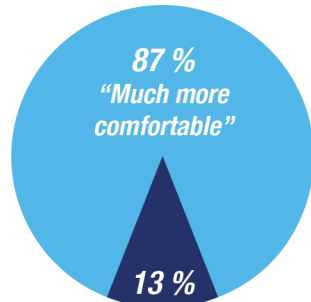Goldpad Manual Assist Technique (MAT) Patient Comfort vs. Previous

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WHY HOLD PRESSURE WHEN GOLDPAD CAN?

Let Goldpad assist you in obtaining and maintaining hemostasis. The clinical studies demonstrate that Goldpad 28cm is safe and effective in reducing active compression time in femoral artery cannulation following diagnostic and interventional procedures. Goldpad simplifies pre- and post- hemostasis management of the access site. It reduces demands on the staff, maximizes valuable resources and enhances patient comfort.

Goldpad 28 cm reduces active compression time.

- Diagnostic Case: 5 minute minimum active compression time
- Interventional Cases: 10 minute minimum active compression time

Low complication rate

- Goldpad has a lower mean complication rate vs. manual compression.

	Diagnostic	Interventional
Active Compression Time	5 min	10 min
Passive Compression Time	120 min	240 min
Total Compression Time	125 min	250 min

Total Major Complication Rate	
Goldpad Patient	1 %
Manual Compression	2.4 %

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UPS İlaç Tıbbi Malzeme ve Kozmetik San. ve Tic. Ltd. Şti.