Primagel®

Lubricant Steril Jel (Lidokainli)
Lubricant Sterile Gel (with Lidocaine)

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PRODUCT DESCRIPTION

It is a disposable, sterile and water-soluble gel used in a syringe in the field of urology mainly for the purpose of lubrication in order to facilitate the administration of the therapeutic devices such as clean intermittent catheterization, permanent catheters or dilatation; diagnostic devices such as urethroscopy or cystoscopy and to facilitate the pain management with the effect of local anesthesia and to reduce the risk of infection thanks to its disinfectant properties.

CONTENT OF THE PRODUCT

Lubricant Sterile Gel with Lidocaine: Lubricant Sterile Gel with Lidocaine HCL is a sterile medical device with local anesthetic and antiseptic agents, which are used for administration on the urethra, prior to catheter or medical devices entering the urethra. Through the use of the medical device content consisting of 97% deionized water by volume and 2% Lidocaine HCL (anesthetic) and 0.25% Chlorhexidine di Gluconate (antibacterial) and through the use of 0.75% Hydroxyethylcellulose (HEC), the sliding performance of the product is increased (See Table.1). The sterile aqueous product is applied to the urethra and is used before implementing a tube or medical device (catheter, etc.) in the body cavity. Thus, the medical device surface that will enter into the urethra is smoothed. It is aimed to reduce the pains and aches of the patient due to the local effect of the Lidocaine HCL in the content of the gel. In addition, the use of Chlorhexidine Gluconate, which is one of the substances in the content of the Lubricant Sterile Gel with Lidocaine, helps the gel to gain antibacterial properties (antiseptic effect) (Lidocaine has antibacterial properties, too). The risk of infection is reduced by its antiseptic properties. The lubricant property (lubricity) is achieved by hydroxy ethyl cellulose (HEC). In this way, the patient relaxes and it reduces the spasticity or discomfort caused by iatrogenic injury.

The product is a disposable, sterile and water-soluble gel in a syringe. The product Lubricant Sterile Gel with Lidocaine is used to facilitate the manipulation of the catheter/other medical devices within the body, to prevent the friction-induced damage on the urethra, and to enable the medical devices to function painlessly by anaesthetizing the local area of application.

Table.1 - Content and Raw Materials of the Lubricant Sterile Gel with Lidocaine;

Description of Material	Volume of Raw Materials in 100 ml
Lidocaine	2 ml
Chlorhexidine digluconate	0,25 ml
Natrosol (Hydroxyethyl Cellulose-HEC)	0,75 ml
Deionized Water	97 ml

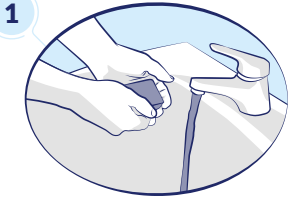
REASON FOR CLASSIFICATION

The product is within the scope of Class III according to the Annex IX, Rule 7 of 93/42 EEC Medical Device Directive and in conformity with the assessment method in Annex II.

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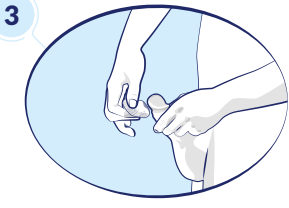
Get Into Position

- Choose a comfortable position, either sitting or standing as suggested by your physician.

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
Prepare for Application

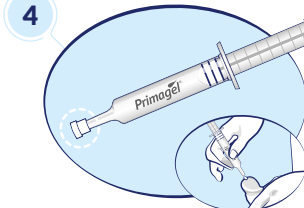- Disinfect the urethra at the head of the penis.
- Raise the penis position as shown in the picture.

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
Prepare Primagel

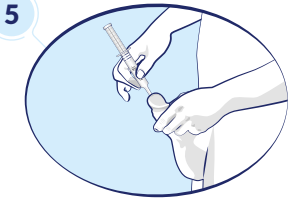- Pick up the sterile ready-to-use Primagel.
- Remove the cap of the syringe with your hand.
- Apply 1 ml of the steril lubricant gel to the urethral opening.

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Instill The Gel

- Carefully insert the cone of the syringe into the urethra.
- Slowly instill the lubricant.

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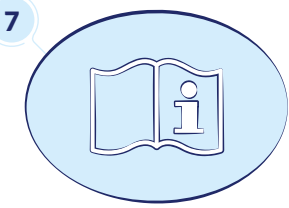
Exposure Time

- The lubrication property of the gel becomes instantly effective.
- The minimum expected time for local anesthetic lubricant gel to be effective is between 5 and 10 min.

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User Manual

- The gel is ready for use in the application areas specified in the definitions section of the user manual included in the box after application.

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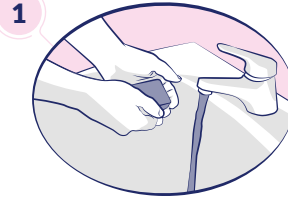
Disposal and Hygienic Precaution

- Dispose of the syringe and the remaining gel as medical waste after the application.
- Wash your hands after application.

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
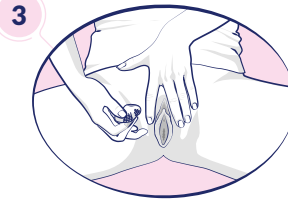
Prepare for Application

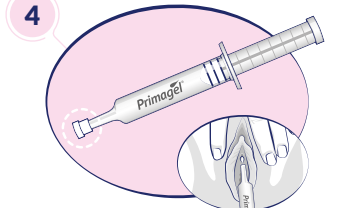- Disinfect the urethra opening.
- Use your fingers to spread the labia to clearly expose the urethral opening.

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